# Salvafix®

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SECURE THERAPEUTIC  
RESTRAINT SYSTEM

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Memory *PRO* · Standard · Slim · Transit

Patient's rights and needs in the 21<sup>st</sup> century require innovative and effective solutions to guarantee the safety and respect of their dignity. SALVAFIX is the result of constant research in which many healthcare professionals are involved and it is adapted taking into account the recommendations of several international agencies for surveillance and control of medical devices. SALVAFIX is recognized and recommended by the healthcare professionals as one of the most reliable, safe and comfortable preventative systems of therapeutic restraint and as respectful to the patient.

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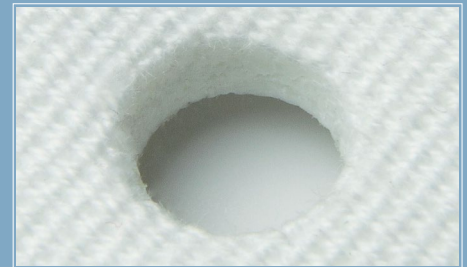
## Labeling

Reinforced structure in all of the sensitive areas where there are increased efforts.

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Ultrasound sealed eyelet holes to avoid tears.

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Smoothly padded edging and reinforced that prevent patient injuries.

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The stainless steel eyelets are highly resistant to tear and moisture, providing more strength.

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Velcro® to facilitate a fast and secure application.

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Edging color: blue (sky) to upper parts and brown (ground) to lower parts.

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# CERTIFIED PERMANENT ANTIMICROBIAL PROTECTION

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The SVF3500 and SVF3505 magnetic opening buttons are made of a material that incorporates the Biomaster 408 additive in its composition, which provides antibacterial properties throughout the useful life of the product. It has been tested according to ISO 22196:2011 standards in addition to environmental controls and is effective against fungi, mold and 99.99% of bacteria, even those resistant to antibiotics such as Staphylococcus aerial (MRSA) and Enterococcus resistant to vancomycin (VRE).

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## Magnetic opening

Apertura magnética

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For general use. Resists up to 150 kg. Included as standard.

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### Waterproof

Robust magnetic button whose function is not altered in wet conditions. Resists up to 300 kg.

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### Aluminium / Aluminio\*

For intensive use (psychiatric, emergency, etc.) Resists up to 400 kg.

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### The "Blue Key"

Magnetic key with **centering ring** for easy positioning on the button and speed up the opening. With **neodymium magnet**, up to 6x more powerful than a ferrite magnet and more durable because they do not lose particles. Ensures quick release of the button.

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## Mechanical opening

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### Mechanical / Mecánico

The system of highest resistance (up to 600 kg.) Water resistant.

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### Mechanical key / Llave mecánica

Made of stainless steel. Designed for quick and safe opening of the mechanical button. It works under any conditions.

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# Salvafix®

## Memory PRO

Makes the application easier and increases the security and control, avoiding risks for the patient due to too strong tightening or for the medical staff due to having it too loose.

The **alphanumeric coding** is a patented system to make the product application easier, quicker and safer. Allows tracking of abdominal measurement changes and memorizes the suitable alphanumeric setting for each patient.

The **Controlled Mobility System** allows to widen or restrain, in a secure way, the grade of mobility according to the person requirements.

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The SALVAFIX Memory Pro abdominal control belts include an articulated system (except size XL) that allows the patient to turn on to either side, be incorporated or sit on the bed. Installation in bed is simple. The sidebands prevent the patient from rotating around the axis of the belt. Perineal band prevent the patient from rotating inside the belt and from the risk of suffocation by sliding down.

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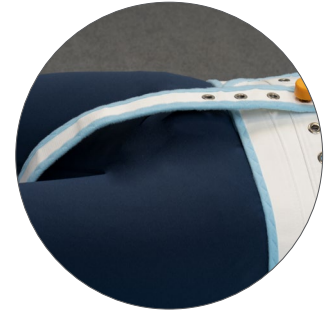
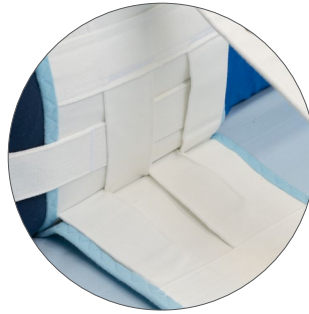
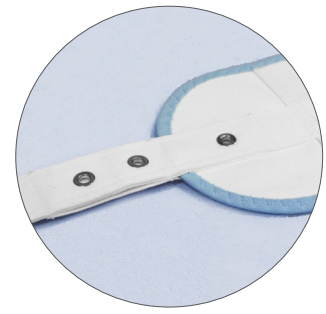
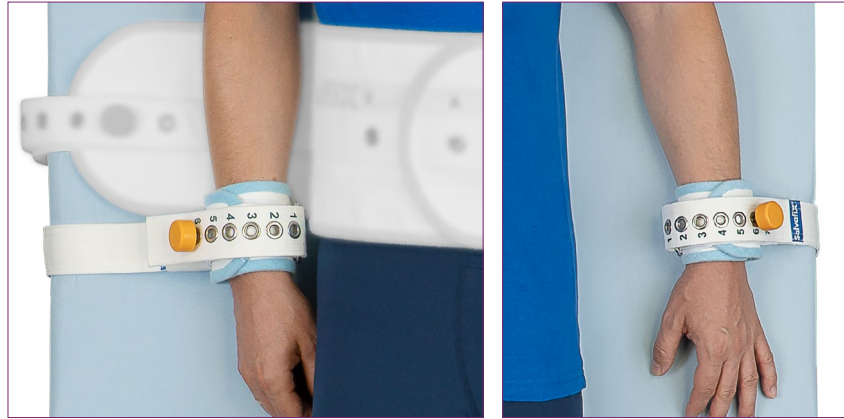
2

3

The SALVAFIX Memory Pro wristbands allow to secure the patient's hands to the bed. These are fixed to the bed structure by a long adjustable band.

1. The **Controlled Mobility System** works combined with the abdominal belt, passing the wristband band through the lower belt loop, thus limiting the movements outside the perimeter of the bed.
2. The inside of the wristbands is made of a soft touch padding, preventing pressure sores or abrasions.
3. The wristbands contains an area with Velcro® that allows faster and tight application, increasing system security.

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## Ankle bands

Tobilleras

The SALVAFIX Memory Pro ankle bands allow to secure the patient's feet to the bed. The set consists of a strap fixed to the bed structure and two independent ankle bands that are fixed to the bed by locks.

1. The **Controlled Mobility System** incorporated to the ankle bands allows to adjust the levels of restraint to the clinical needs of the patient.
2. The inside of the ankle bands is made of a soft touch padding, preventing pressure sores or abrasions.
3. The ankle bands contain an area with Velcro® that allows faster and tight application, increasing system security.

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## Harness with thorax band

*Arnés con banda para tórax*

Prevents the patient from slipping vertically up, and is highly flexible in its application. A band around the thorax prevents the patient from slipping and loosen the harness. This product comes with a transversal strap which prevents any violent or immediate incorporation so that the patient cannot injure himself or others. For non violent patients, this harness can be used without the transversal strap. This way the patient obtains a higher grade of mobility and is able to sit up on the bed but not to slip up.

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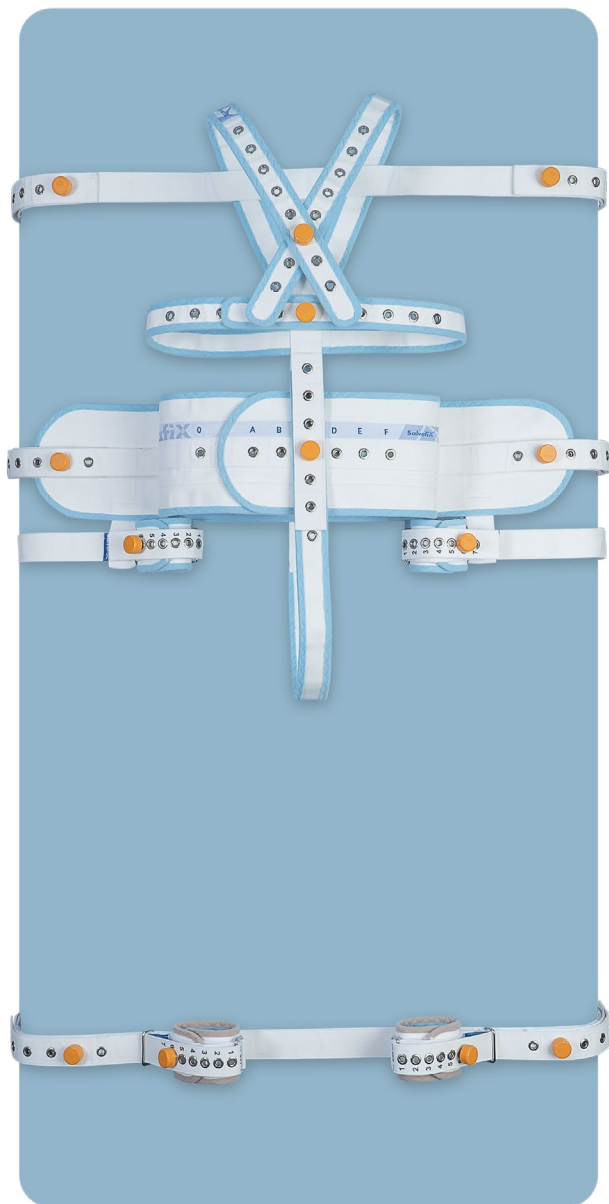
## Complete set

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The complete set is the global solution of secure restraint for bedridden patients. It permits having the maximum level of security of the patient and the medical staff, as bed restraint is not an easy issue. As it consists of individual elements, it can be used as required by the patient.

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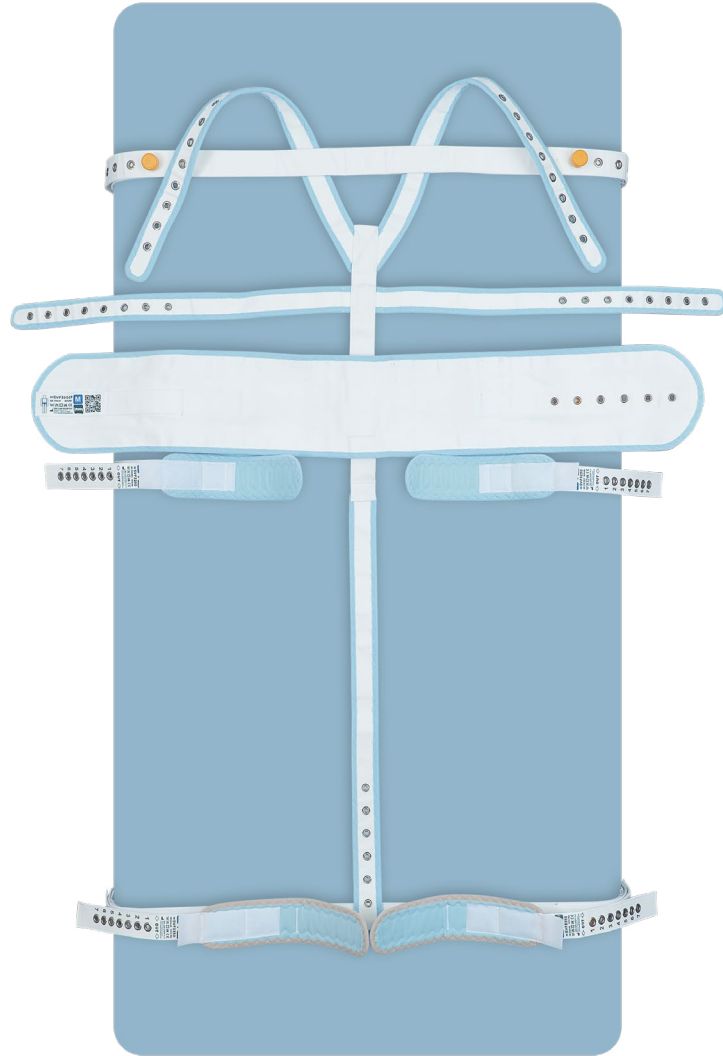
# Salvafix®

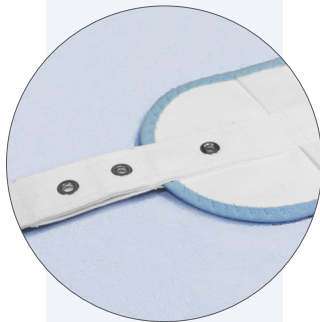
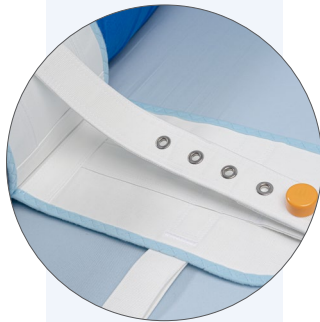
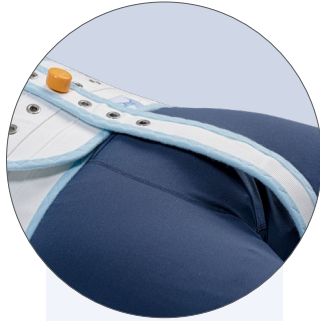
## Standard

The STANDARD range enables  
a secure and versatile restraint.  
Products are made with highly  
resistant materials, have a functional  
design and are comfortable.

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## Abdominal belt

Cinturón abdominal

The abdominal restraint belt can be installed before or after the patient lies down on the bed. The lower part, with fixing straps and two tabs, has to be fixed to the bed (always on moving parts) structure; the upper part has to be placed around the patient's waist. The lateral restraining bands must always be applied and used and they are intended to control or prevent the patient from rolling in excess to the right or to the left, and let the appropriated postural changes be made.

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Short

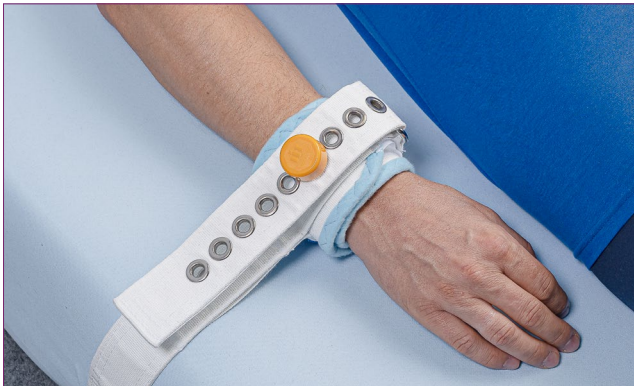
Cortas

Its particular design permits a comfortable application only to the belt tab.

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Long

Largas

Designed to be applied without a belt, anchored to the bed frame.

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Monoblock

Wristbands sewn to the bed band in a single piece.

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
## Universal Duo

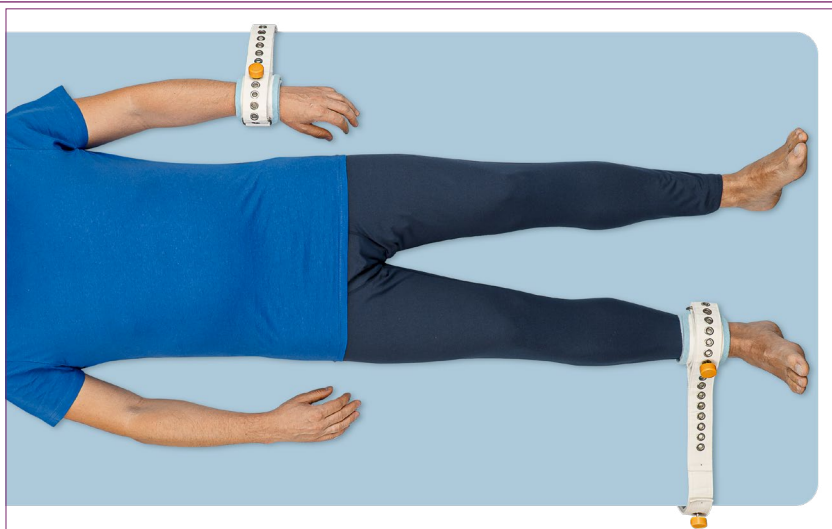Versatile and universal size restraint valid for use on wrists and/or ankles.

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## Ankle bands

### Tobilleras

The SALVAFIX Standard ankle bands allow to secure the patient's feet. The comfortable softly padded covering eliminates the injuries caused by pressure or abrasions. The Velcro® prevents slipping of the overlapped parts.

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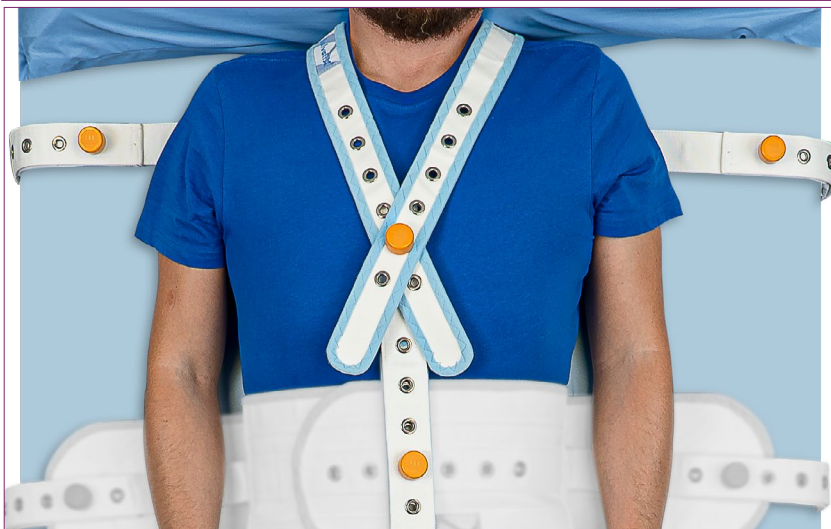
## Harness

Arnés

The fastening thoracic harness is used to prevent the patient from slipping vertically.

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## Thigh bands

Musleras

The SALVAFIX Standard thigh bands allow secure the patient's legs.

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## Complete set

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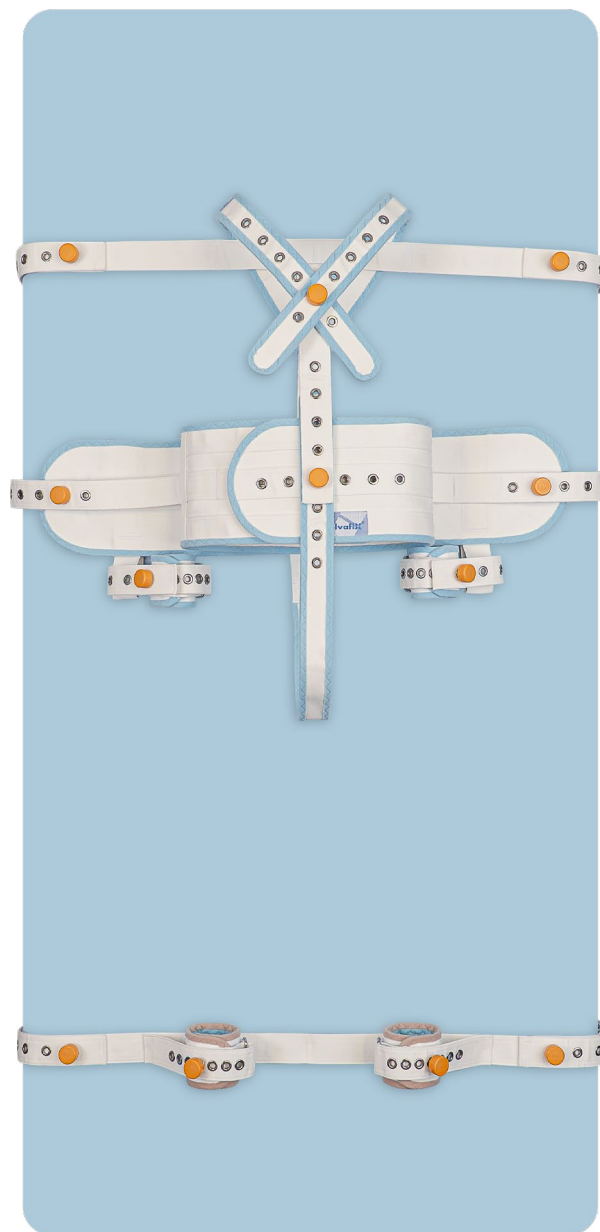
This set permits having the maximum level of security of the patient and the medical staff. As it consists of individual parts, it can be used depending on the need at that moment.

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# Salvafix®

## Slim

Allows a secure and adapted restraint for slim patients on the bed, wheelchair, chair or armchair. Ideal for home care due to its security, convenience and simplicity.

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## Belt for bed

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Belt with lateral strips to prevent the patient from rotating around the axis of the belt. The perineal band prevents the patient from rotating inside the belt and from the risk of suffocation by sliding down.

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## Belt for wheelchair, chair or armchair

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# Salvafix®

## Transit

Restraint system specifically adapted  
for the transfer of patients on  
stretchers or through ambulation.

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## Harness for stretcher

### Arnés para camilla

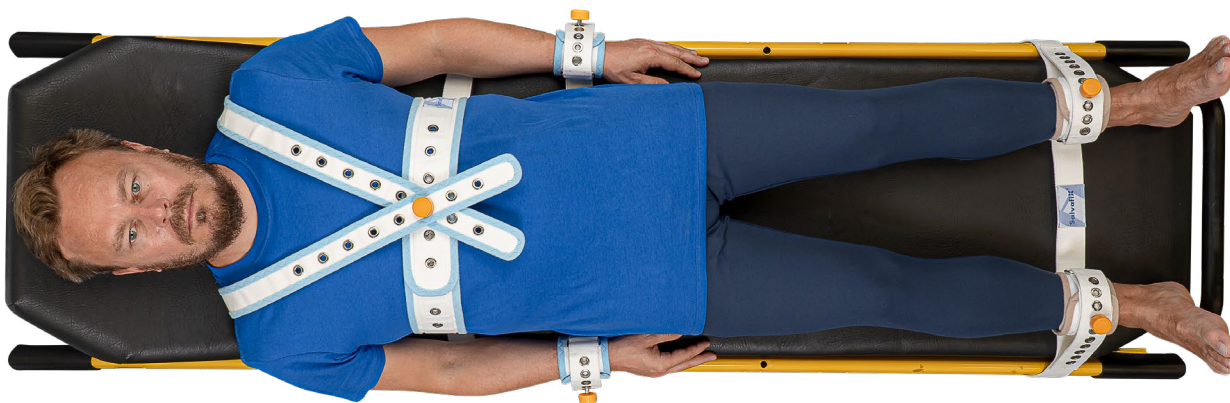
The thoracic fastening harness is used to avoid the patient from moving vertically. A blue edge sewn on the borders of the straps gives it a soft finish and creates a soft touch padded surface. Its particular monoblock design permits an easy application on the stretcher.

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## Wristbands for stretcher

### Muñequeras para camilla

Wristbands adapted to the stretcher assembly. Its monoblock design permits an easy application on the stretcher.

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## Ankle bands for stretcher

### Tobillos para camilla

Ankle bands adapted to the stretcher assembly. Its monoblock design permits an easy application on the stretcher.

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## Wristbands for ambulation

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Set of wristbands sewn onto the belt band. Maintains patient's arms in a comfortable position and avoids aggression, self-injury or any manipulation of locks.

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## Ankle bands for ambulation

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Set of ankle bands sewn on a band that connects them. Allows the patient to walk in short steps.

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## Binding band for wristbands and ankle bands

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- 1 binding band / banda de unión
- 1 lock / cierres

## Complete set for ambulation

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Consists of a set of wristbands sewn onto the belt band, a set of ankle bands and a binding band. Maintains patient's arms and legs in a comfortable position and avoids aggression, self-injury, any manipulation or access to the locks by the patient. Allows the patient to walk in short steps as well as to sit.

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Emergency box with cutter included  
*Caja de emergencia con cutter incluido*

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Code/ Código  
SVF3475

## Demo case

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Demonstration set consisting of a robust branded briefcase containing a complete Salvafix kit.

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**evo**  
TRAINING  
INSTITUTE

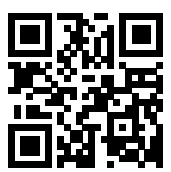
<https://evotraining.eu>

SALVAFIX is one of the safest and most effective restraint systems on the market, thanks to its complete training program for distributors and professional users, allowing them to use the product with agility and security guarantees. Both health center-based and distance training are offered, with an extensive range of audiovisual equipment and learning resources. Continued improvement of the SALVAFIX system makes continued learning the best vehicle for achieving excellence in the product usage.

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For more detailed information on all products see brochure instructions and safety information included with each item. You can also download it using this QR code.

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